Instructions For Use Kitazato ET Catheter

Revision#8.0

CE 2797

EN : General Information

Refer to the label on the product or package before use.

Symbols on the Label

	Do not reuse		Batch code
	Use by		Re-order number
	Consult Instructions for use		Manufacturer
	Sterilized using ethylene oxide gas		EC representative
	Sterilized using irradiation		Do not re-sterilize
	Do not use if package is damaged		

Intended use

The Kitazato ET catheter is used to place sperm, oocyte or embryo into the uterine cavity through the cervix, the myometrium or the abdominal.

Caution

Bleeding may occur as a result of trauma due to insertion of the catheter through the cervix. Do not press the catheter strongly.

Contraindications

- No reuse. It may cause infection when reuse.
- No re-sterilization.

Instructions for use

- (1) Position the patient in the lithotomy position. Insert vaginal speculum to expose cervix. Check the condition of the cervix and the endometrium, and decide the insert length and the angle of the catheter and the guide before use.
Caution : Use a stylet guide in case the cervix bends and it seem to be difficult to pass the cervix.
- (2) Wash the vulva and the vagina with sterilized saline and dry them.
- (3) Connect a syringe to the connector of the catheter and aspirate embryo or gamete to the catheter under microscope.
- (4) Insert the catheter to the cervix.
Caution : Use a guide or a stylet guide as assistance when the catheter does not go through the cervix by itself.
- (5) Check the catheter reached to the uterus under ultrasound. Slowly press the plunger of the syringe and expel the embryo or the gamete into the uterine cavity.
- (6) Carefully remove the catheter.
- (7) Check for retention of embryos.
- (8) Take the vaginal speculum out and keep the patient at rest.

Cautions for use

- (1) Confirm adaptability with the other medical equipment to use together beforehand.
- (2) Do not wash the inside of the catheter with sterilized saline before infusing the catheter with embryo or gamete. It may cause troubles for aspirating embryo or gamete.
- (3) Read an Instructions For Use before use by all means.
- (4) Do not use the product that it was past an expiration date.
- (5) Immediately use after opening the package.
- (6) If the product has abnormalities that its package has been broken or it has been damaged before opening it, do not use it.
- (7) Do not jack up forcibly or reprocess.
- (8) Use the product by operator that has seasoned knowledge about procedure for the product.
- (9) Do not use the product except the transfer of embryo or gamete.
- (10) Use the product aseptically. Carry out enough precautionary measures for bacteria.
- (11) Throw the product away promptly after having used it.
- (12) Do not refurbish.
- (13) Store away from high temperature, high humidity, the water wet.

Warranty

The manufacturer is not liable for any damage resulting from misuse or incorrect operation or failure to follow the cautions or instructions contained in this Instructions For Use or the instruction manual for any compatible medical equipment.

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Kitazato Corporation Tokyo Office