Instructions For Use Kitazato IUI Catheter

Revision#9.0

EN : General Information

Refer to the label on the product or package before use.

Symbols on the Label

	Do not reuse		Batch code
	Use by		Re-order number
	Consult Instructions for use		Manufacturer
	Sterilized using ethylene oxide gas		EC representative
	Sterilized using irradiation		Do not re-sterilize
	Do not use if package is damaged		

Intended use

The Kitazato IUI catheter is used for the introduction of washed spermatozoa into the uterine cavity through the cervix.

Cautions

Bleeding may occur as a result of trauma due to insertion of the catheter through the cervix. Do not press the catheter strongly.

Contraindications

- No reuse. It may cause infection when reuse.
- No re-sterilization.

Instructions for use

- (1) Position the patient in the lithotomy position. Insert vaginal speculum to expose cervix. Check the condition of the cervix (the angle of the uterine ante- flexion and retroflexion), and decide the insert length of the catheter.
- (2) Wash the vulva and the vagina with sterilized saline and dry them.
- (3) Connect the 6% tapered syringe to the connector.
- (4) Load the catheter with washed sperm solution.
- (5) Insert the catheter to the cervix. If necessary, make the tip of the shaft to the suitable angle to pass the cervix smoothly.
- (6) After insertion to the uterine cavity, slowly press the plunger of the syringe and expel the sperm solution or the washed spermatozoa into the uterine cavity.
- (7) Carefully remove the catheter.
- (8) Discard the syringe and catheter as medical waste.

Cautions for use

- (1) Confirm the specification (length) of the catheter.
- (2) Read an Instructions For Use before use by all means.
- (3) Do not use the product that it was past an expiration date.
- (4) Immediately use after opening the package.
- (5) If the product has abnormalities that its package has been broken or it has been damaged before opening it, do not use it.
- (6) Do not jack up forcibly or reprocess.
- (7) Use the product by operator that has seasoned knowledge about procedure for the product.
- (8) Do not use the product except the transfer of embryo or gamete.
- (9) Use the product aseptically. Carry out enough precautionary measures for bacteria.
- (10) Throw the product away promptly after having used it.
- (11) Do not refurbish.
- (12) Store away from high temperature, high humidity, the water wet.

Warranty

The manufacturer is not liable for any damage resulting from misuse or incorrect operation or failure to follow the cautions or instructions contained in this Instructions For Use or the instruction manual for any compatible medical equipment.

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Kitazato Corporation Tokyo Office